



Black & White Printers

**UP-X898MD / UP-991AD**

**STILL IMAGE CAPTURE**

**STORE STILL IMAGES on USB Flash Drive**



**EASY : JUST PLUG IT IN !**

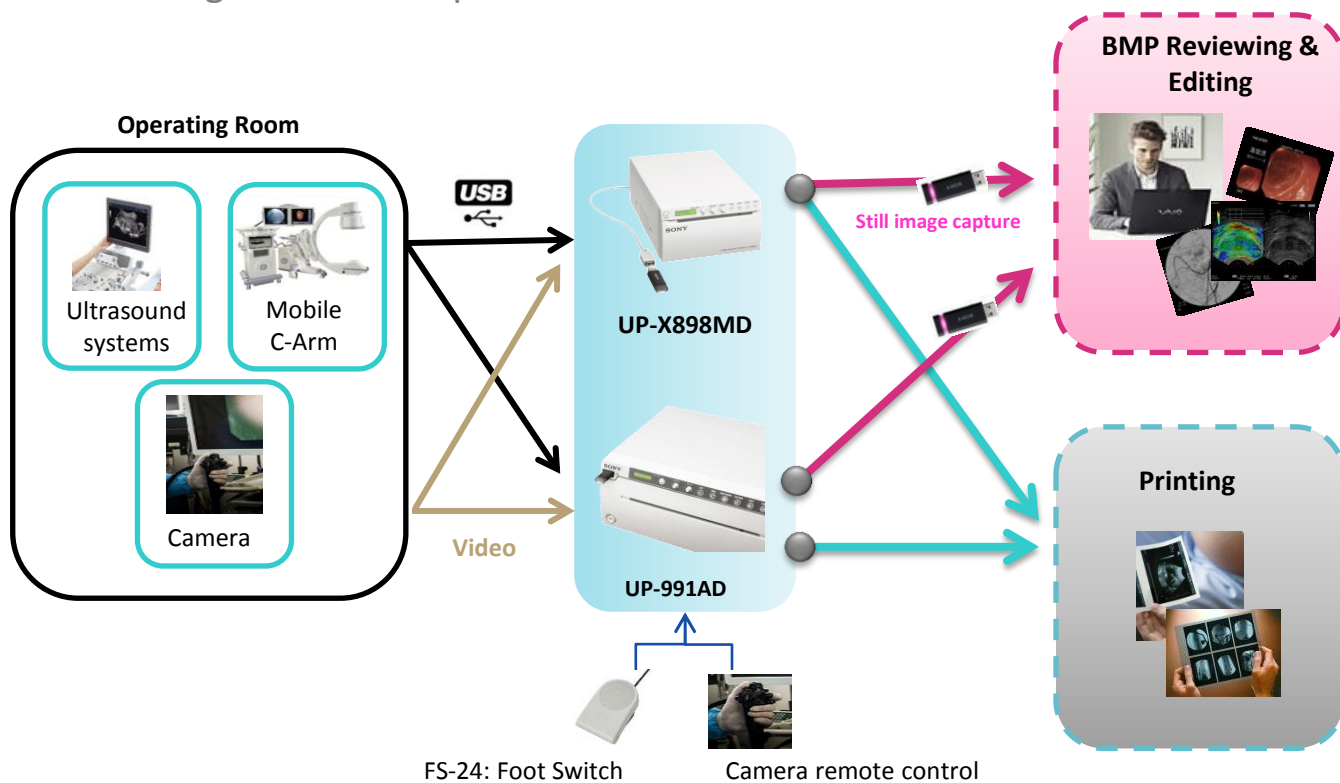


Store images on a USB Flash drive without additional manual steps required. Remove the flash drive and insert them into your laptop or PC to transfer the images for use in patient reports, presentations, or publications.



**Key Benefits**

- Still Image Storage on an external USB Flash Drive.
- Easy Digitalization of your video input
- Capture & Store image only (UP-991AD)
- Easy capture via Foot Switch or camera remote control button

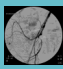
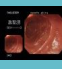
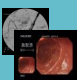
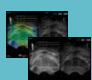
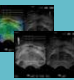
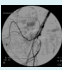
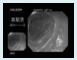
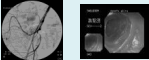
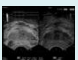
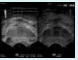

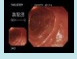
**Configuration example**



# Still Image Capture feature Comparison chart

	UP-991AD	UP-X898MD
USB connector	Front 	Back Extension cable provided 
Capture & Print	✓	✓
Capture only	✓	
Storage from Video input	✓	✓
Storage from Digital input	✓	✓
Storage of Color images	✓	
Recording Status	✓	✓
Format	Bmp	Bmp
File Name (automatic incrementation)	UP_0000000 to UP_9999999	UP_000001 to UP_9999999
Folder Name SN : serial number of the printer	\\SONY\\UP-991AD\\SN	\\SONY\\UP-X898MD\\SN

The below chart shows how the image is recorded into the USB flash memory depending on printer settings and input source.

	UP-991AD			UP-X898MD	
	B&W Video Input 	Color Video Input 	Digital Input 	Video Input 	Digital Input 
Storage & Printing					
Storage only			N/A	N/A	N/A



### Help instructions of Still Image Capture feature

available in the menu :

- Select item : PRT.HELP
- Select setting : HELP.5

⇒ Instructions from user manual are printed

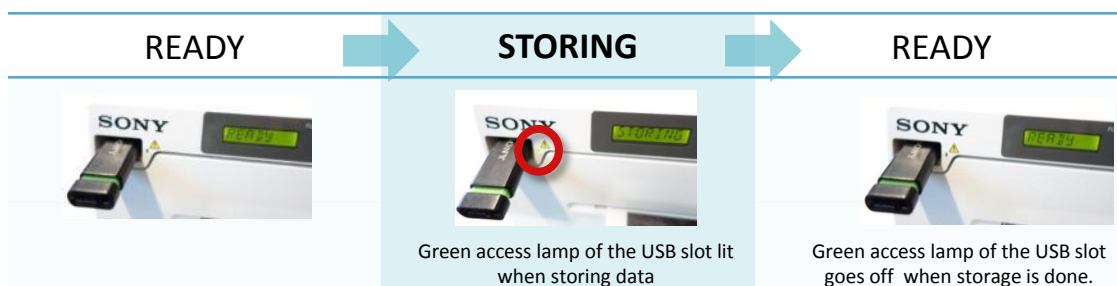
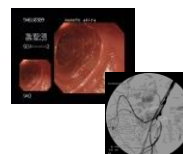
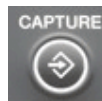




# Still Image Capture feature UP-991AD Settings Information

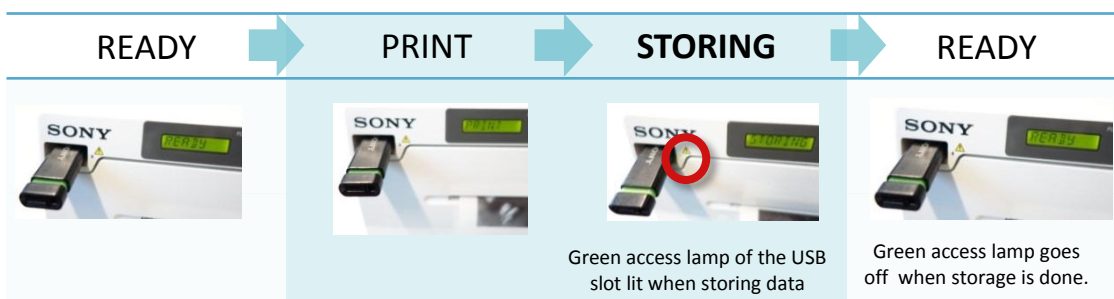
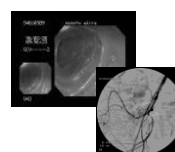
## ■ Configuration of Storage ONLY

- Storage by pressing **CAPTURE** button
- 1. In the MENU, Select VIDEO,
- 2. Select setting STR.KEY ⇒ **STR : CAPT**
- ONLY possible when the printer is connected to a video source
- Storage of Color or Black& White images :
  1. In the MENU, select VIDEO
  2. Select setting COLOR
    - ⇒ **COLR : ON** to store color images
    - ⇒ **COLR : OFF** to store black & white images
- When MULTI is ON, each image is stored in a separate file
- Status on LCD displays:



## ■ Configuration of Storage and Print (default setting)

- Storage by pressing **PRINT** button
- 1. In the MENU, Select VIDEO,
- 2. Select setting STR.KEY => **STR : PRNT**
- With video and digital input
- Storage of Black& White images only
- To get the best image quality with color input
  1. In the MENU, select VIDEO
  2. Select setting COLOR ⇒ **COLR : ON**
- When MULTI is ON, the layout with the 2, 4 or 6 images is stored as one image (one file).



When AUTOCUT is ON, "CUTTING" is displayed after "PRINT" message, "STORING" message may not be seen



# Still Image Capture feature UP-X898MD Settings Information

## ■ Configuration of Storage and Print

- When USB flash drive is plugged in, images are automatically stored when PRINT button is pressed or when PRINT function is sent remotely :



- When MULTI is ON, the layout with the 2 images is stored as one image (one file).



## Notes for UP-991AD and UP-X898MD

- Remove the USB flash drive ONLY when 'READY' is displayed
- "NO SPACE" is displayed when the USB flash drive is full
- To store images with the correct date and time, adjust them in the menu :
  - Select item : CONFIG
  - Select setting : CLOCK => Set date and time (Greenwich time by default)
- Image saved on the USB flash drive cannot be printed again
- Connection of USB Hard Drive not supported
- Using a USB Flash Drive manufactured by Sony Corporation is highly recommended to guarantee the storing function.
- Connection of the USB connector to the USB flash drive through a hub is not guaranteed.
- Compatible with USB Hi-Speed
- Corresponding file systems are FAT16 and FAT32